# C3249W

CAMILLA FIJA / STATIONARY TABLE / TABLE FIXE

## INSTRUCCIONES - INSTRUCTIONS

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etiqueta CE

## **CAMILLA FIJA INSTRUCCIONES**

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Espuma grado de firmeza II - Medio

## **STATIONARY TABLE INSTRUCTIONS**

Ecopostural thanks you for buying our products. The item you have just purchased has been manufactured according to high quality and security standard. Follow the below-mentioned instructions carefully in order to be able to make use of it for a long time. Click on to the **TECHNICAL INFORMATION** section in our website **[www.ecopostural.com](http://www.ecopostural.com)**

**Ecopostural tables have been conceived to support the patient during an examination and/or treatment session by healthcare personnel or under his supervision in those fields such as physiotherapy, rehabilitation and related disciplines.**

|                                                                                                                                                                                                                   |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Our company will decline any responsibility for a product damage occurred if ignoring our operating instructions. We will not be liable for any accident or harm resulting from an undue or negligent use.</b> |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

## INDEX

### Symbols \*

1. General instructions
2. Operating instructions
  - 2.1 Set Up
  - 2.2 Load capacity
  - 2.3 Sections adjustment
3. Maintenance
  - 3.1 Frame
  - 3.2 Upholstery
4. Guarantee
5. Technical features
6. FAQ about motors and upholstery

## SYMBOLS

Follow the instructions of this user manual.

Read carefully before the first use and observe the operating instructions and the preventive security measures.

**Warning or caution:** This sign alerts about a possible hazard which must be overcome so as to avoid:

- a. Serious or even potentially deadly injuries for the patient.
- b. Mild or moderate severe injuries for the patient.
- c. Material damages to the equipment.

If the use of this device causes or contributes to the presence of an undesired event, such as death or serious injury to the user, the manufacturer MUST be informed immediately via email

[vigilanciaps@ecopostural.com](mailto:vigilanciaps@ecopostural.com) or the distributor. In the European Union, the user must also notify the competent authorities of the member state in which he is located.

Avoid sitting down on the table headrest.

This product meets the Medical Device Regulation from the European Parliament and the European Council dated 5th April 2017: **(EU) 2017/745**

States the item is a Medical Device

Manufacturer's name, address and manufacturing date.

Refers to the model number

Unique Device Identifier

Manufacturing date

Lot number

Equipment with double isolation IEC 60417-5172

Expendable electrical components that can be recycled

Stands that the electric and electronic components of the device can be recycled and must be disposed of separately.

Waste separation of used products and packaging allows the materials to be recycled and used again. Re-use of recycled materials helps prevent environmental pollution and reduces the demand for raw materials.

\* according to models

## 1) GENERAL INSTRUCTIONS

- Please read carefully the operating and safety instructions before using our products. These instructions must be adhered to. Keep the instructions booklet for further check-ups.
- Familiarize yourself with the precise use of the item.
- If the table is handled by unqualified people in an inappropriate way or not according to the intended purpose, the use of the examination table may be dangerous. Be careful when handling the table as there is a risk of being trapped by the mobile parts and because of sudden movements.
- Regularly check the table entirely to find any sign of malfunction or damage.
- Every single part or component of the table follows a thorough quality control thus ensuring a smooth finish. Nevertheless, proceed with caution when handling the table because there might be any corner or protruding part which might result unintentionally harmful.
- Worn or damaged pieces must be solely replaced by original pieces by the manufacturer depending on the table model.
- All components parts or accessories should be fitted or fixed in according to the special manufacturer's instructions. Any spare / replacement component part different to the manufacturer's could be dangerous for the people involved in the table handling.

**\* Notice: The use of non original pieces or the alteration of the item will incur in a loss of validity of the manufacturer's guarantee, instructions and Declaration of Conformity.**

**Ecopostural reserves the right to modify its products and make changes in technical data. Actual dimensions and weights may vary from the values indicated in this catalogue:  
dimensions  $\pm 5\text{cm}$  / weight  $\pm 5\text{kg}$  / movements  $\pm 10^\circ$ .**

**Our products are handcrafted and made-to-order upon the customer preferences.**

**Consequently, we won't accept any cancellation, return or refund of any Ecopostural product because there's been a previous choice for a specific size, colour...**

**The design of the different sections and tilting angles obeys to the standards of specific profesional areas such as the physiotherapy, rehabilitation and related disciplines, hence the specific dimensions conforming to these practices.**

**ATTENTION: Be careful, as there is a risk of entrapment between moving and/or fixed parts and due to unexpected movements. Only operate the stretcher when there is no possibility that any part of the body could be trapped and injured while moving the stretcher up/down or activating the tilt of the component modules.**

## 2) OPERATING INSTRUCTIONS

### a) **SET UP**

The table is adjusted at the minimum height when being packed. Follow the "*Height adjustment*" instructions for a tight-fitting.

Right below the upholstery you will find a box containing the wooden wedges aimed at shimming the table.

***Notice:*** in certain floor surfaces the table legs might leave marks after a prolonged contact. Check the technical specifications of the floor manufacturer to avoid any trouble.

### b) **LOAD capacity**

This item has been tested to support a **MAXIMUM STATIC WEIGHT of 135 Kg** evenly distributed all over.

Absolutely avoid leaning on the very edge for support as it may tilt over.

If pressure on a reduced area exceeds the allowed weight, the table upholstery could split up. Keep the

patients from getting on by their knees. The maximum weight limit of every table section is 50 kg.

### c) HEIGHT adjustment

The table is equipped with **extendable legs** allowing the height adjustment according to the following instructions:

- Unscrew the fixing bolt till the leg is completely released.
- Lift one side of the table for an easy access to the leg. In case this leg is shimmed with a wooden wedge you will have to remove it as well. The inside leg will slide down gently. Choose the required position (among the 4 available heights) and fix it by matching the holes.
- Tight the fixing bolt back and shim the inside leg with the necessary wedges up to a maximum of 3 per leg.
- Follow the same procedure in every leg.

In case any of the inside legs loosens completely, simply tight it back on by making the slot match the fixing bolt.

**\* Notice: before using the table, make sure all legs are fixed at the same height.**

### d) SECTIONS adjustment

The **head-/backrest** section is operated by rack (2)

The **central section** is operated by rack (2)

(2) Rack: Lift the section up to the required position and the rack will position itself automatically. To lower the section, first raise it up to a certain extend or completely (as appropriate) in order to release the rack.

*\* The hinges on both sides must be properly fixed and positioned at the same height.*

**NEVER adjust any of the table sections WHILE THE PATIENT IS LEANING ON.**  
**Safely position your FINGERS AND HANDS AWAY from the upholstery edge** to avoid any accidental jamming.

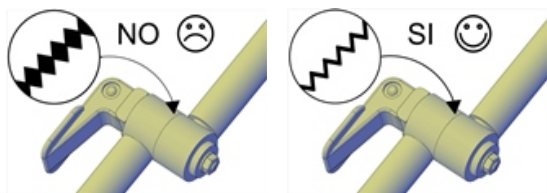
### e) ACCESORIES Adjustment

The table you've bought includes the **FACE CRADLE A4430**.

It is **stored** underneath the upholstery and fixed by means of 3 clamps. To remove the face cradle simply hold the metal frame and pull.

To **install** the face cradle, insert the bolts through the holes provided at the edge of the table.

You can **adjust** the cradle tilt by loosening the grips, then place it at the required position and tighten the grips.

*\* Ensure that the toothed pieces fit together correctly before tightening them again.*

## 3) MAINTENANCE

### a) Wooden FRAME

For a proper operation of the table, you should **do the dusting** regularly with a cloth, especially at the moving parts.

Periodically **check** couch visually for any signs of damage and make sure that all bolts, nuts and other fasteners are tight.

*\* The repeated exposure of the table to high levels of humidity is not recommended.*

### b) UPHOLSTERY

A wet cloth should be enough to do the dusting. Anyway, if the leather is not properly clean you can use a

smooth sponge soaked with neutral soap water. Dry immediately afterwards. To prevent moisture from entering the seams, do not wash with abundant water. It is highly recommended to use a towel cover like the A4469, A4459 or A4460 to protect the upholstery from sweat, massage creams or oils. Use soapy water to clean the stains. **Avoid using alcohol, any solvent or stain remover** as the vinyl could be seriously damaged. The use of infrared lamps might cause a cracking of the artificial leather. You should then put a cotton sheet or towel on the couch.

**\* For further details please CHECK THE ENCLOSED UPHOLSTERY INSTRUCTIONS.**

## ECOPOSTURAL GUARANTEE

All our equipment is covered by a **3 years** guarantee, from the original invoice date, against any manufacturing flaw or mechanical defect. It will be thus necessary to keep the invoice or any other purchasing document as proof of guarantee.

Wear and tear of the artificial leather will not be covered by the guarantee.

The following conditions will invalidate your guarantee: any damage or deterioration caused by misuse (e.g. inappropriate leather cleaning), by an undue unwrapping with a cutter (e.g. upholstery rips and tears) or by accident.

In those cases where the guarantee applies, Ecopostural will repair or replace the faulty part free of charge and return it at our expense within the EU territory.

For full details of the guarantee, please contact your supplier.

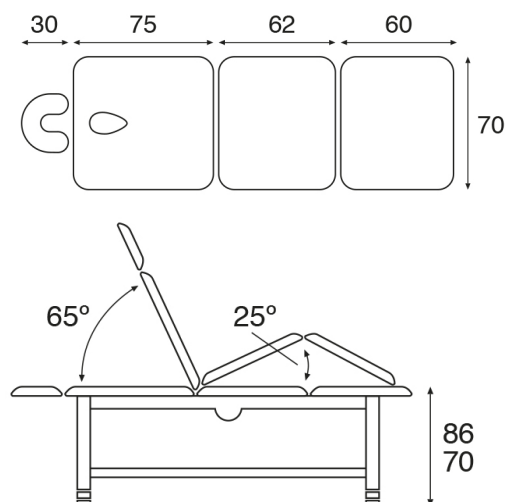
**PRODUCT**  **MANUFACTURED BY ECOPOSTURAL, S.L.U.** ESB12433991  
Avda. Enrique Gimeno, 27 - 12006 Castellón de la Plana, **SPAIN**  
[www.ecopostural.com](http://www.ecopostural.com) / [info@ecopostural.com](mailto:info@ecopostural.com) / +34 964 20 01 53

 The disposal of this product is subject to a specific EC Directive and it must be followed accordingly, as shown by the crossed skip symbol. Therefore, at the end of its useful life, you will have to apply to a specialized waste disposal centre.

Separate Collection of used products and packaging allows materials to be recycled and used again. Re-use of recycled materials helps prevent environmental pollution and reduces the demand for raw materials.

## TECHNICAL FEATURES C3249W

### General characteristics

3 section wooden stationary table wenge finish.

Standard upholstery dimensions: 70x198 cm (M66)

Optional upholstery dimensions: 80x198 cm (M76)

Recommended maximum weight: 135 Kg

Height variation: 70-86 cm

Multi-positioning internal system manually operated height.

Made in high quality beech wood.

Lower wood shelf for storage included.

Standard backrest type T33 - Square backrest. Oval breather hole and bung included.

Detachable multi-position face-rest (A4430) included as standard.

Head-/Backrest adjustable manually, by rack.

Central section adjustable manually, by rack.

Approx. weight: 50 Kg

### Upholstery Features

Fire-resistant Class M2 artificial leather, EC compliant.

Sanitized® anti-microbial and antibacterial protection.

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